ENGLISH
MIXING AND APPLICATION
<ul style="list-style-type: none">Extrude equal amounts of base and catalyst Dispense LIFE just prior to using Do not interchange caps Wipe nozzle of each tube before replacing cap Mix with small spatula for 10 seconds - until streak free Place immediately into cavity with Kerr ballpoint applicator Wipe applicator tip before replacing into mix when successive applications are made

Handling Features
<ul style="list-style-type: none">Adequate work time on the pad Quick set in the mouth Flow characteristics for controlled placement Carveable

CAUTIONS
<ul style="list-style-type: none">High temperature and humidity accelerate set time. Store at ambient conditions. Keep nozzle clean and replace the cap tightly after use. When operating under "dry field" conditions (such as rubber dam) and using the acid-etch technique, there may be insufficient moisture on the cavity floor to effect the set of LIFE as promptly as desired. In such cases, place a small drop of water on the cavity liner. This will cause the material to set quickly.

- Some bonding agents (chemically cured enamel bonding agents, chemically cured dentin/enamel bonding agents and some light activated dentin/enamel bonding agents) possess a high level of benzoyl peroxide and have the potential to cause discoloration of LIFE and LIFE FAST SET.
- It is recommended that you test materials' compatibility by placing the bonding agent and/or dentin bonding agent on the surface of a set LIFE sample (on the mixing pad) to determine systems' compatibility.

PROPERTIES
<ul style="list-style-type: none">Maintains pulp vitality in cases of exposures and near exposures Promotes formation of reparative dentin Insulates pulp against thermal shock Protects pulp from permanently placed fillings Has a high early strength (withstands packing pressures) Resistant to acid etching Does not inhibit polymerization of acrylic or composite restoration or accessories.

WORK TIMES
Work times on pad:
Life Minimum of 3 minutes at 75°F (24°C) and 50% humidity
Life Fast Set Minimum of 2 minutes at 75°F (24°C) and 50% humidity

LIMITED WARRANTY LIMITATION OF KERR'S LIABILITY
Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. Such advice does not expand Kerr's limited warranty or relieve the dentist of testing Kerr's products to determine their suitability for the intended uses and procedures. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product.

In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.
<p>EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.</p>

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