Instructions for Use	Bruksanvisning
Gebrauchsinformation	Brugsanvisning
Mode d'emploi	Käyttöohjeet
Istruzioni d'uso	Bruksanvisning
Instrucciones de uso	Productinformatie
Instruções de Uso	Οδηγίες Χρήσεως

- Finishing, polishing, and high gloss polishing system
- Finier-, Polier- und Hochglanzpoliersystem
- Système de finition et de polissage au brillant
- Sistema di rifinitura, lucidatura e lucidatura a specchio
- Sistema de acabado, pulido y pulido a alto brillo
- Sistema de acabamento, polimento e polimento de alto brilho
- Finisherings- och poleringssystem
- Finisherings-, polerings- og højglanspoleringssystem
- Finer-, poler- og højglanspolersystem
- Järjestelmä viimeistelyyn, kiillotukseen ja erittäin hohtavaan kiillotukseen
- Fineer-, polijst- en hoogglanspolijststysteem
- Σύστημα λιανσης και στιλβωσης

For dental use only!

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein

Ivoclar Vivadent AG, FL-9494 Schaan

557414/0706/WE3/G

ivoclar
vivadent®
clinical

Instructions for Use


Description

Astropol is a polishing system that consists of four differently shaped instruments. The corresponding working procedure requires 3 steps.

Working steps:

1. Finishing
2. Polishing
3. High gloss polishing

Shapes:

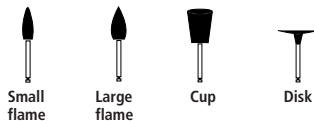Composition

The polishers consist of silicone rubber, silicon carbide, aluminium oxide, titanium oxide, and iron oxide. Astropol HP additionally contains diamond dust. The handles are made of stainless steel.

Indication

Astropol is used for polishing direct, tooth-coloured restorations, as well as ceromers.

Contraindications

If a patient is known to be allergic to any of the above components, the polishers should not be used.

Side effects

None known to date.

Application

- **1st Step: Finishing with Astropol F (grey)**
With the Astropol F finisher, excesses are removed and a smooth surface can be achieved.
- **2nd Step: Polishing with Astropol P (green)**
Polishing with Astropol P results in a very delicate surface finish.
- **3rd Step: High gloss polishing with Astropol HP (pink)**
With the microfine Astropol HP diamond finisher, a high gloss can be achieved quickly and easily. Do not apply pressure.

Important notes

- Recommended speed: 7500–10'000 rpm
- Use in conjunction with a water spray
- Avoid excess heat development
- Astropol is autoclavable up to 135°C / 275 °F.
- Polishing pastes are not necessary

Warning

Do not inhale grinding dust.

Storage

- Store Astropol at 2–28 °C (36–82 °F)
- Shelf life: see date of expiration
- Do not use Astropol after the indicated date of expiration

Store out of the reach of children.

For use in dentistry only.

Date information prepared: 02/2006

Rev. 1

Ivoclar Vivadent AG

FL-9494 Schaan/Liechtenstein

This product has been developed solely for use in dentistry. It should be used strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent AB
Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.