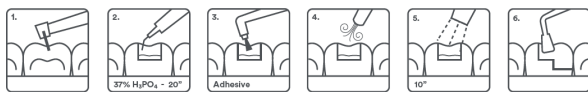
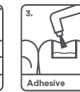
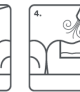
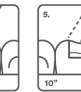
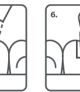
FICHA TÉCNICA

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TECHNICAL DATA SHEET

DENTAL ADHESIVE TOTAL ETCH PROCLINIC EXPERT		
REFERENCES	36599: 1 unit 36598: 2 units	
MANUFACTURER	SDI Germany GmbH Hansestrasse 85 D-51149, Cologne Germany	
CLASSIFICATION	Medical Device, Class IIa, Directive 93/42/EEC.	
DESCRIPTION	Fluoride releasing single component dentin/enamel total - etch adhesive system, designed for direct bonding. The product completely and homogeneously infiltrates the hybrid layer to ensure superior bonding to the tooth.	
COMPOSITION	Acrylic monomer, Acetone, Fluoride, Stabilizer.	
PACKAGING	Package with 5ml bottle.	
SHELF LIFE	3 years.	
INTENDED USE	For use only by dentists. I) All direct restorative composites. II) Compomers. III) Composite, ceramic, metal, porcelain repairs. Bonding surface requires pretreatment with a silane primer.	
INSTRUCTIONS FOR USE	<div style="display: flex; justify-content: space-around; align-items: center;">       </div> <ol style="list-style-type: none"> 1. Isolate and prepare tooth. 2. Apply Etch as per manufacturer's instructions. Wash thoroughly. Remove excess water. Keep moist. 3. Apply the product to saturate all internal surfaces. 4. Blow gently with dry, oil free air for 2 s to evaporate solvent. Leave surface glossy. 5. Light cure for 10 seconds. 6. Apply composite as per manufacturer's instructions. 	
PRECAUTIONS	Avoid prolonged contact with skin/soft tissues as it may cause inflammation or sensitisation. Discontinue use if irritation occurs or if have known acrylic resin allergies. When not in use, replace cap.	
HANDLING AND STORAGE CONDITIONS	Store in a cool dry place between 10 and 25°C, away from direct sunlight. Protect from moisture. Keep out of the reach of children	