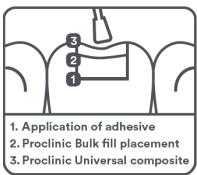
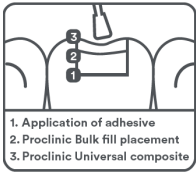
FICHA TÉCNICA

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TECHNICAL DATA SHEET

BULK FILL COMPOSITE PROCLINIC EXPERT		
REFERENCES	36618	
MANUFACTURER	SDI Germany GmbH Hansestrasse 85 D-51149, Cologne Germany	
CLASSIFICATION	Medical Device, Class IIa, Directive 93/42/EEC.	
DESCRIPTION	Light cured, radiopaque, high strength and high aesthetic composite restorative system designed to simplify everyday shade matching requirements for the clinician.	
COMPOSITION	Methacrylate, Dimethacrylates, Silicon dioxide, Inorganic filler.	
PROPERTIES	The product can be cured to a maximum of 5mm in 20 seconds with an LED curing light or 2 x 20 seconds with a halogen curing light. Offers superb handling, sculptability and easy polishing.	
PACKAGING	Package with 4g syringe.	
SHELF LIFE	3 years..	
INTENDED USE	For use only by dentists. Posterior restorations. Core build ups. Splinting. Sandwich technique with glass ionomer.	
INSTRUCTIONS FOR USE	<div style="display: flex; align-items: flex-start;"> <div style="flex: 1;">  <p style="font-size: small;">1. Application of adhesive 2. Proclinic Bulk fill placement 3. Proclinic Universal composite</p> </div> <div style="flex: 2; padding-left: 10px;"> <ol style="list-style-type: none"> 1. Isolate tooth. Prepare cavity using standard technique for composite resin restorations. Note: In cases where pulp protection is necessary, use a calcium hydroxide liner. 2. Apply a dentine / enamel bonding agent as per manufacturer's instructions. 3. Place Bulk Fill composite. 4. Light cure for 2 x 20 sec / 5 mm using a LED curing light or any other light curing device (470nm wavelength). 5. Finish restoration using standard technique. </div> </div>	
PRECAUTIONS	Avoid prolonged contact with skin / soft tissues as it may cause inflammation / sensitisation. Discontinue use if irritation occurs or if have known acrylic resin allergies. Use at room temperature. When not in use, replace cap.	
HANDLING AND STORAGE CONDITIONS	Store in a cool dry place between 10 and 25°C, away from direct sunlight. Protect from moisture. Keep out of the reach of children	