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**INTENDED USE**

Definitive direct relining of dental prostheses

**INDICATION**

Suitable for all PMMA-based plastic materials.

Production of permanently soft relinings directly at the chairside or indirectly in the laboratory.

Removal of pressure points in the mandibular and maxillary areas.

For post-damming the A line in case of adhesion problems and flabby ridges.

As an elastic margin on maxillary dentures and for securing the position of the dentures by increased adhesion.

For elastic covering of sharp-ridged alveolar processes.

For covering implant areas and as overdentures.

**PATIENT TARGET GROUP**

Persons being treated in the context of a dental procedure.

Proclinic Expert soft relining material automix system polish: Do not use for the treatment of pregnant/breastfeeding women and of children.

Catalyst component contains alkyl stannosiloxanes.

**INTENDED USERS**

Dentist, dental technician

## **1. PREPARATION OF THE DENTURE**

**Removal of the old relining (Fig. 1)** In general, an existing relining must be removed completely. Grind off the base to be relined by approx. 1 - 2 mm, depending on the situation; at the vestibular, lingual or dorsal edges, incorporate a shoulder approx. 2 - 3 mm in height with a step of at least 1 mm; round off the borders of the denture and clean the denture thoroughly.

**Degreasing (Fig.2)** Degrease with pure, medical alcohol and allow to dry.

**Application of the Proclinic Expert soft relining material primer automix system (Fig. 3):** All surfaces ground and to be relined to which the material is required to adhere must be evenly and completely covered with **Proclinic Expert** soft relining material primer automix system . Let **Proclinic Expert** soft relining material primer automix system dry for approx. 1 minute. Close the bottle immediately after use. Saliva, moisture or monomer liquids prevent adhesion of **Proclinic Expert soft relining material automix system** to the denture.

## **2. MIXING**

**Proclinic Expert soft relining material automix system :** Application of the mixing jet / dispenser

Insert the cartridge according to special instructions. Remove and discard cartridge closure; do not reuse! Slightly squeeze material until it flows evenly out of both openings. Mount the mixing cannula on the cartridge and squeeze out the required material. Leave the mixing cannula on the cartridge as a closure after each use.

## **3. DIRECT CHAIRSIDE RELINING**

**Coating of the denture (Fig. 4):** Prepare the denture as described under item 1. Apply **Proclinic Expert soft relining material automix system** cartridge material by means of a dispenser (see item 2), taking care to avoid bubbles, to the vestibular, lingual or dorsal borders, respectively, covering the entire surface evenly. Then coat the remaining prosthetic base plate completely (layer thickness: min. 2 mm). All denture areas that come into contact with the gingiva must be coated.

**Perform the functional movements (Fig. 5):** Then place the denture in the patient's mouth. Let him/her close the mouth in normal occlusion. The curing process is completed after approx. 5 minutes. Then remove the relined denture from the mouth, and trim and paint it as described in item 5.

#### **4. INDIRECT METHOD IN THE LABORATORY**

Invest the denture to be relined into a flask with counter. For easier removal of the denture, cover the teeth with silicone before pouring the counter, e.g. with **Flexistone® Plus**. Remove the denture from the flask and then prepare it as described in item 1. Coat the model with a standard plaster-plastic insulation (alginate). As described under item 3, apply mixed **Proclinic Expert soft relining material automix system**, respectively, taking care to avoid bubbles, evenly over the entire surface of the denture area to be relined (layer thickness: approx. 1 mm more than previously milled away). Close the flask with the counter and leave it under a press for approx. 30 minutes with light pressure, or in a pressure pot at 40 - 45 °C / 104 - 113 °F in a water bath for 10 minutes. Then remove the denture from the flask and trim and paint it as described in item 5.

#### **5. FINAL TRIMMING AND PAINTING**

**Smoothing of the borders:** First remove coarse excess with a sharp scalpel or scissors (Fig. 6). Then use suitable rotating tools (e.g. **Molloplast®** cutters, pre-polishers and grinding sleeves, from approx. 8,000 RPM) with light pressure to clean and smooth the cut borders (Fig. 7). **Polish (Fig. 8):** **Proclinic Expert soft relining material automix system polish** should be used only to smooth the ground functional borders. Open the aluminum seal with a spatula and plug on pipettes. Pipette both liquids at a mixing ratio of 1:1 onto a mixing block, mix homogeneously with a spatula, and apply with a brush; allow to dry for approx. 5 minutes at room temperature.

#### **6. CLEANING RECOMMENDATION FOR THE PATIENT**

Thoroughly brush and rinse the relined denture several times a day with a soft toothbrush and neutral soap under running water; cleaning bath: max. 10 minutes.

## **SAFETY INFORMATION**

- ▶ Only for the specified intended use by trained specialists.
- ▶ Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- ▶ Wear suitable personal protective equipment (protective gloves, goggles, face mask) when finishing the cured material.
- ▶ Adhesion is ensured only on roughened surfaces that have been coated with Proclinic Expert soft relining material primer automix system!
- ▶ Avoid eye contact with catalyst, rinse eyes immediately with plenty of flowing water if necessary, consult ophthalmologist immediately.
- ▶ Apply Proclinic Expert soft relining material primer automix system over the entire surface!
- ▶ All denture areas to be relined that will come into contact with the gingiva must be evenly and completely coated with Proclinic Expert soft relining material automix system otherwise adhesion problems may occur!
- ▶ Curing times must be complied with!
- ▶ Proclinic Expert soft relining material primer automix system , Proclinic Expert soft relining material automix system polish: Keep away from sources of ignition.
- ▶ Refer to the relevant safety data sheet for hazard and safety information.

## **NOTES**

- ▶ DETAX shall not be held liable for any damage caused by misuse.
- ▶ Read and understand the safety data sheet!

**For users and/or patients:**

Any serious incident that has occurred in connection with this product must be reported immediately to [incident@detax.com](mailto:incident@detax.com) and to the competent public authority of the country in which the user and/or patient resides.

**CONTRAINDICATION**

Do not use this material in case of known allergies to any of the ingredients, or contact allergies. Undesirable effects are not to be expected in case of proper application. However, as a matter of principle immune reactions such as allergies or irritations cannot be ruled out. In case of doubt, we recommend performing an allergy test before application.

**DISPOSAL**

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

**Storage:**

15 °C  
59 °F

25 °C  
77 °F

**Processing:**

At  $23 \pm 2$  °C /  $73 \pm 3$  °F ,  
 $50 \pm 5\%$  of rel. humidity

\* from start of mixing at  $23 \pm 2$  °C /  $73 \pm 3$  °F,  $50 \pm 5$  %

of rel. humidity. Higher temperatures shorten, lower temperatures extend the indicated times.

<b>Technical specifications</b> DIN EN ISO 10139- 2 type A, soft	<b>Proclinic Expert soft relining material            automix system</b>
<b>Mixing volume</b>	50-ml cartridges
<b>Product color base/catalyst</b>	red-brown / white
<b>Mixing time</b>	not applicable (Automix system)
<b>Open time*</b>	approx. 1 min 30 s
<b>Curing time*</b>	approx. 6 min 30 s
<b>Retention time in the mouth</b>	approx. 5 min
<b>Final hardness</b>	approx. 28 Shore A
<b>Mixing cannula</b>	 Automix2, pink

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detax GmbH
PROCLINIC S.A.U.