**FICHA TÉCNICA**

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### TECHNICAL DATA SHEET

<b>ROOT CANAL SEALER PROCLINIC EXPERT</b>		
<b>REFERENCES</b>	78550	
<b>MANUFACTURER</b>	<b>B.J.M. Laboratories Ltd.</b> 12 Hassadna St. Industrial Park, Or-Yehuda 6022011 ISRAEL Tel. 972-74-7000111	
<b>CLASSIFICATION</b>	Medical Device, Class IIa, Directive 93/42/EEC.	
<b>DESCRIPTION</b>	Two-paste epoxy-amine resin root canal sealer in a dual auto-mixing syringe. <b>Characteristics:</b> <ul style="list-style-type: none"> <li>- Excellent sealing capability.</li> <li>- High radiopacity.</li> <li>- Non-cytotoxic material.</li> <li>- Long term stability.</li> <li>- Moderate flexibility preventing cracking of fully cured material.</li> <li>- Low shrinkage.</li> <li>- Good flowability.</li> <li>- Conforms to ISO 6876.</li> </ul>	
<b>COMPOSITION</b>	<b>Paste 1:</b> Bisphenol-A-(epichlorhydrin) and epoxy resin, Zirconium Oxide, Silica, Iron Oxide pigments. <b>Paste 2:</b> Poly (oxyethylene, oxypropylene) diamine, Silica, Zirconium Oxide, Antibacterial agent.	
<b>PACKAGING</b>	10g Automix Syringe, 10 Automix Syringe Mix Tips and Inta Oral Tips, 1 Mixing Pad.	
<b>SHELF LIFE</b>	2 years	
<b>INTENDED USE</b>	Permanent obturations of root canals combined with root canal points.	
<b>INSTRUCTIONS FOR USE</b>	Prepare the root canal, clean and dry before use of Proclinic Expert Root Canal Sealer. Remove the cap by turning it 90° counterclockwise and pulling it off. Bleed the syringe prior to use. Attach the mixing tip to the syringe and then turn the tip 90° clockwise. Auto mixing syringe ensures proper homogeneous mixing	

	<p>of two pastes and allows direct application of the sealer into the root canal.</p> <p>After the application, the mixing tip must be removed by rotating it 90° counter clockwise and pulling it off. Then the auto mixing syringe must be closed with the original cap.</p>
<b>PRECAUTIONS &amp; CONTRAINDICATIONS</b>	<p>Hypersensitivity to epoxy resins and amino groups. Attention: do not to use in case of patient who has allergenic sensitivity to epoxies and amines.</p> <p>In case of contact with eyes or skin: wash immediately with plenty of water. Any excess of material must be removed with cotton applicator and then washed with water followed with suction.</p> <p>For external use only! If swallowed, drink plenty of water. If any sickness symptoms seek medical assistance.</p>
<b>HANDLING AND STORAGE CONDITIONS</b>	<p>To be stored at temperature from 4 to 24°C.</p> <p>The syringe must be closed after use with original cap only.</p>