FICHA TÉCNICA

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SAFETY DATA SHEET

MTA PROCLINIC EXPERT		
REFERENCES	78551: White MTA cement.	
MANUFACTURER	ANGELUS INDÚSTRIA DE PRODUTOS ODONTOLÓGICOS 101, Waldir Landgraf Street Londrina, Paraná 86031-218, Brazil	
CLASSIFICATION OF THE PRODUCT	Class IIa Medical Device, according to Regulation (EU) 2017/745.	
DESCRIPTION	Bioceramic endodontic cement composed of several mineral oxides. It is constituted by thin hydrophilic particles.	
SPECIFICACIONES	MTA: 1 g; Distilled water: 3 ml. Non sterile product.	
PACKAGING	1 unit MTA, 1 unit distilled water, 1 measuring spoon	
SHELF LIFE	3 years	
INTENDED USE	Indicated in cases of root canal lateral and furcation perforations, internal resorption, reverse root filling, pulp capping, pulpotomy, apexification, and apexogenesis.	
INSTRUCTIONS FOR USE	<ol style="list-style-type: none"> 1. Sterilize a glass slab, a metal spatula and all instruments for the insertion of Proclinic Expert MTA. 2. Mix for 30 seconds the content of 1 sachet of Proclinic Expert MTA (or 1 spoon of Proclinic Expert MTA) with 1 drop of distilled water. The mixture should be homogeneous and with a consistency similar to wet sand; 3. Place the cement on the selected site with a sterilized amalgam carrier or other appropriate instrument; 4. Condense the cement with instruments such as amalgam condensers, a number 1 spatula or absorbent paper points moistened with distilled water. <p><i>(*) Important: If Proclinic Expert MTA is not used immediately after mixing, its dehydration can be prevented and the working time increased by covering the mix on the glass slab with a wet gauze. If the mixed MTA dehydrates, it must be discarded.</i></p>	

WARNINGS & PRECAUTIONS	<ul style="list-style-type: none"> • Use eye protection, mask and gloves when handling Proclinic Expert MTA. In case of eye or skin contact, wash abundantly with water; • Only open the sachet or bottle immediately prior to use. The Proclinic Expert MTA powder is very sensitive to humidity. Close the bottle tightly after each use; • Do not use Proclinic Expert MTA to fill a root canal. Its viscosity is inadequate for this procedure and very difficult to remove in case of a reentry; • Do not use Proclinic Expert MTA in areas of the tooth in contact with the gingival sulcus or it will be completely dissolved; • Do not use MTA in areas with exposure to light in order to avoid darkening of the dental structure; • Only use Proclinic Expert MTA after remission of acute signs and symptoms of the endodontic disease. The acidic pH of endodontically compromised sites (lesions) prevents its setting reaction; • Apply Proclinic Expert MTA carefully. Similar to other endodontic cements, it is resorbed if extruded. However, the excess of any cement may impair the healing process; • Do not use MTA on patients with reported sensitivity to any of its components. <p>For exclusive use of the dentist.</p>
HANDLING AND STORAGE CONDITIONS	<p>Store in a cool dry place, away from direct sunlight. Protect from moisture and excessive heat.</p>