FICHA TÉCNICA

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TECHNICAL DATA SHEET

PROCLINIC EXPERT UNIVERSAL ADHESIVE		
REFERENCE	78564	
MANUFACTURER	P.L. Superior Dental Materials GmbH 22607 Hamburg Germany	
CLASSIFICATION	Medical Device, Class IIa, according to Directive 93/42/EEC concerning medical devices.	
DESCRIPTION	Universal adhesive based on methacrylates for adhesive restorations.	
COMPOSITION	Modified methacrylates, MDP, catalysts, ethanol stabilizers, photoinitiators. It does not contain BPA derivatives.	
SPECIFICATIONS	Recommended curing time (light cure): 20 s Specific gravity (20 °C): 1,07 g/cm ³ Adhesive strength to dentine (light cure, 20 s): >15MPa Adhesive strength to enamel (light cure, 20 s): >15MPa	
PACKAGING	Package with one unit of 5 ml.	
INTENDED USE	Professional dental use. Used for the adhesive restoration for the self-etch technique, selective enamel etching or total-etch technique.	
SAFETY INFORMATION	<p>In some cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.</p> <p>Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore, for pulp protection areas close to the pulp should be covered with a thin layer of calcium hydroxide material.</p> <p>Incompatibility with other materials: Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Do not store the material in proximity of eugenol containing products. Don't let the material come into contact with materials containing eugenol.</p> <p>Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates</p>	
SHELF LIFE	2 years from the date of manufacture.	
STORAGE CONDITIONS	Do not store above 25 °C. Avoid storage in direct sunlight.	