## FICHA TÉCNICA

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## TECHNICAL DATA SHEET

<b>CERAMIC PRIMER PROCLINIC EXPERT</b>		
<b>REFERENCE</b>	78579	
<b>MANUFACTURER</b>	<b>B.J.M. Laboratories Ltd.</b> 12 Hassadna St. Industrial Park- Or-Yehuda 6022011 ISRAEL Tel. 972-74-7000111	
<b>CLASSIFICATION</b>	Medical Device class IIa, according to Directive 93/42/EEC concerning medical devices.	
<b>DESCRIPTION</b>	Surface conditioner that improves adhesion between ceramic surfaces and resin materials.	
<b>COMPOSITION</b>	Ethanol, Water, Organic acid, Silane coupling agent, 4-META.	
<b>PACKAGING</b>	Bottle containing 5 ml.	
<b>INTENDED USE</b>	Preparation of ceramic crowns, veneers and inlays prior to cementation. Preparation of fractured ceramic crowns for repair with resin materials.	
<b>SHELF LIFE</b>	2 years.	
<b>INSTRUCTIONS FOR USE</b>	<ol style="list-style-type: none"> <li>1. Clean ceramic surface with alcohol and dry well with oil-free air.</li> <li>2. Open Proclinic Expert Ceramic Primer and dispense 1-2 drops into mixing well, replace bottle cap immediately.</li> <li>3. Use applicator to apply generous amount of primer onto ceramic surfaces.</li> <li>4. Wait 1 minute.</li> <li>5. Dry gently for 20 seconds with oil-free air.</li> <li>6. Continue with cementation.</li> </ol>	
<b>SAFETY INFORMATION</b>	Avoid contact with skin, oral soft tissues and eyes. If accidental contact occurs, rinse with copious amount of water and seek medical attention immediately.	
<b>HANDLING AND STORAGE CONDITIONS</b>	Store at room temperature below 24°C (75°F).	