English

## Kolor + Plus™

Color Modifiers for Tooth Shading, Characterizations, and Masking Tool  
Coloration

KOLOR + PLUS™ is a light color, viscous liquid resin color modifier and opaque kit. It contains eight different color modifiers and two different opaques. These modifiers were designed to be used under or between layers of Herculite XRV/Prodigy/Revolution or other resin restorative. They are not designed to be used on the surface of restorations. They offer the ultimate in complete color creativity for the discerning dentist and laboratory technician.

### Versatility and Ease of Use

- Create an infinite number of hues by mixing two or more modifiers.
- Adjust the opaques with modifiers to alter the chroma/hue and the value.
- Adjust the chroma or intensity by mixing modifiers or opaques with untinted modifier.
- The sequence now varies depending upon the nature of the characterized restoration.
- Mix (if necessary) the modifiers and opaques to be used (see General Suggestions for Kolor + PLUS™).
- Opakers are generally placed to block out or modify the color of the underlying tooth.
- The modifiers and opaques should always be embedded within the resin restorative, except when it is a surface characterization of minimal surface area such as a check or craze line.
- The resin restorative Herculite XRV/Prodigy/ Revolution is placed last.

### General Suggestions for Kolor + PLUS™

1. Mix drops of different modifiers together on a mixing pad or ResinKeeper®.
2. Use a disposable (Kerr Applicator Tips™) brush or ball-point applicator to do the mixing.
3. Add one color to another in very small increments incorporating as little air as possible, until desired coloration is reached.

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### II. Range Change with Opaques

- A1-Light Shade-This light shaded opaque can be used as it is formulated to create Vita® A1 and A2 range. By adding very trace amounts of yellow modifier (to A1 opaque), Vita® A1 and A2 opaques can be created. Adding trace amounts of grey modifier (to Vita® A1 opaque) Vita® C1 and C2 can be created. By adding trace amounts of red modifier (to Vita® A1 opaque), Vita® D3 and D4 can be created. As a specific example of color mixing; take equal amounts of A1 opaque and A3 opaque and mix them well. The resultant shade should be close to A2.

### III. Dark Shades

- This darker shaded opaque can be used as it is formulated to create the Vita® A3 and A4 range. By adding very trace amounts of orange modifier (to Vita® A3 opaque) Vita® C3 and C4 can be created. By adding trace amounts of grey modifier (to Vita® A3 opaque) Vita® D3 and D4 can be created.

### IV. Neutralization

- The opaques are sometimes too opaque and could create a "dead" look. In order to lower the chroma or intensity of the opaques, one can add untinted modifier to the opaque to give it more transparency.

### V. Characterizing the Individual Tooth

- I. Craze Lines-Craze lines are opaque lines found in some teeth where the enamel has internally cracked. This can be recreated in resin by embedding a mylar strip into the surface of Herculite XRV Incisal. The strip should be embedded in the soft incisal about 1 mm deep in the direction you wish the crack to appear. The XRV should be polymerized and the strip pulled out. Vita® A1 opaque or white modifier should be painted over the crease in the surface until it fills it completely. It is then polymerized, contoured and polished.

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