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Please read the instructions for use before using the product.

VITA VIONIC DENT DISC multiColor are acrylic-based, composite milling blanks with an inorganic filler that are used for the CAD/CAM fabrication of denture teeth for removable and partially removable dentures.

**Intended purpose**

VITA VIONIC® DENT DISC multiColor products are resin artificial teeth for dental treatments.

**Patient target group**

No restrictions.

**Intended user**

Dental professionals only: dentists and dental technicians (Rx only).

**Indication**

Indication range:

- Full and partial dentures

**Contraindication**

- Fixed restorations
- In patients with allergies or sensitivities to the ingredients

**Workflow**

Please adhere to the following workflow when working with this product:

1. Impression taking / intraoral scan, bite registration if necessary
2. Where required, fabrication of a model and scanning / impression scanning
3. CAD process (design)
4. CAM process (milling)
5. Reworking (removal, grinding of the restoration)
6. Conditioning of the bonding surface (recommendation: sandblasting with aluminium oxide 250 µm, 2 bar. Blow clean with oil-free pressurized air)
7. Bonding, e.g., using the VITA VIONIC BOND Kit (observe the instructions for use)
8. Individualization/characterization/polishing

**Recommended product combinations**

For recommended product combinations with the VITA system components and corresponding processing recommendations and restrictions regarding the combination possibilities of the product, please refer to Instructions for Use | Full Version.

**Storage/disposal**

Disposal via household waste. Recyclable waste (such as packaging, paper and plastics) must be disposed of via appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

**Technical data**

Physical properties	Unit of measure	Value
3-point flexural strength	MPa	approx. 80
Flexural modulus	MPa	approx. 2800

Chemical composition	Wt%
Silicon dioxide	14 - 15
Polymethyl methacrylate copolymer	84 - 86
Pigments	< 1

**Safety information**

- While work is in progress, wear suitable safety goggles / face protection
- Only perform work under an extraction unit

**Product reliability**

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks and (if applicable) short clinical safety and performance reports (SSCPs) can be found at [https://www.vita-zahnfabrik.com/product\\_safety](https://www.vita-zahnfabrik.com/product_safety).

The safety data sheets can be downloaded at [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com) or requested by fax at (+49) 7761-562-233.

**Disclaimer**

Please note: Our products must be used in accordance with the instructions for use. We accept no liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of this information: 2023-07.

After the publication of this information for use any previous versions become obsolete. The current version can be found under [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com).

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VITA Zahnfabrik has been certified and the following products bear the mark:

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**VITA VIONIC®**  
DENT DISC multiColor

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Instructions for use | short version

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