Refractory Firing Paste

Instructions for use

- DE** Gebrauchsanweisung.
- ES** Instrucciones de uso.
- FR** Mode d'emploi.
- IT** Istruzioni per l'uso.
- PT** Instruções de uso.
- NL** Gebruiksaanwijzing
- PL** Instrukcja postępowania.
- CS** Návod k použití.
- RO** Instrucțiuni de utilizare
- SK** Návod na použitie.
- DA** Brugsanvisning
- SL** Navodila za uporabo.
- ET** Kasutusjuhised.
- HU** Használati utasítás

FEUERFESTE BRENNPASTE

DE

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REFRACTORY FIRING PASTE

EN

Refractory Auxiliary Paste is a ready-to-use paste, ideal for producing customized firing trays. This refractory material securely holds inlays, onlays, veneers, crowns, and bridges, fixing them directly onto the firing tray to ensure stability and precision during the firing process.

1	Apply the ready-to-use fixing paste directly onto the ceramic restoration.
2	a) Secure it onto the firing support. b) For veneers/inlays, fix directly onto the posterior surface. c) It is also possible to fix directly onto the firing tray. Please ensure that the restorations are slightly elevated by filling them slightly more for better results.
3	The fixing paste is ready for immediate use. Apply it after completing the ceramic restoration.
4	Firing.
5	The restorations are easy to handle after ceramic firing. Carefully lift the stump from the firing platform. Sandblasting or cleaning is not required.

Application area and properties:

- Enables stabilization of ceramic works with complex fixation during the firing process in the ceramic oven.
- Quick and easy application with a ready-to-use syringe.
- Ensures that the ceramic does not discolor after firing.
- Easy to remove, no polishing or scraping required.
- Does not require extending the pre-drying time in the ceramic oven.

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